

Using LHSC Pneumatic Tube System for Transporting VersaTREK® Blood Culture Bottles

1. Flip open the (2) latches on the pink REDOX Transporter.
2. Place properly labeled VersaTREK® blood culture bottle(s) securely into the pink REDOX Transporter.




3. Close the REDOX Transporter lid over the bottles and press the (2) latches downward until the transporter is closed securely.



4. Place the pink REDOX Transporter into a large, clear, sealable bag.
5. Put remaining patient labels in the clear bag with the pink REDOX Transporter.



6. Seal the clear bag and place pink REDOX Transporter into a pneumatic tube carrier. **DO NOT PLACE ANY OTHER SAMPLES IN THE PNEUMATIC TUBE WITH THE PINK REDOX® TRANSPORTER**

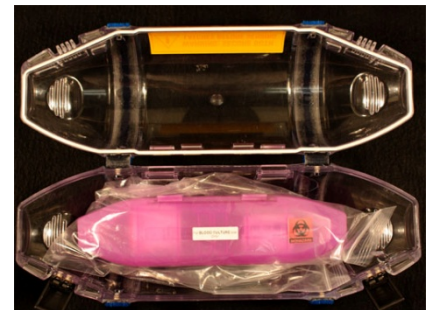

7. Close the carrier. The pink REDOX Transporter is now ready to be sent through the LHSC pneumatic tube system.



DO NOT PLACE BLOOD CULTURE BOTTLES IN LHSC PNEUMATIC TUBE SYSTEM WITHOUT THE PINK REDOX® TRANSPORTER