

L-
ASPARAGINASE

Orderable - LASP

Turnaround Time: 10 days

Alternate Name(s):

Asparaginase Activity
Kidrolase
Oncaspar
Erwinia



Laboratory:
Core Lab



Requisition:

**L-ASPARAGINASE
REQUISITION**



Method of Analysis:
Microplate Reader-Based
Method



Test Schedule:
Referred out Monday-
Thursday



Referred Out Location:
[In-Common Laboratories](#)

Specimen:

4 mL Lavender top Vacutainer tube
OR
6 mL Red top Vacutainer tube

Please avoid gel separator tubes.

Collection Information:

Completed requisition must accompany labeled sample to the lab.
Indicate date and time of last dose and daily dosage.
Minimum volume required for testing is 3 mL

Reference Ranges:

L-Asparaginase: > 0.1 IU/mL

Special Processing:

SRA:
Centrifuge 10 minutes @ RT.
Aliquot into 2 mL aliquot tube and freeze immediately in -70°F freezer.

Storage and Shipment:

Store and send frozen.
Ship on dry ice.