

# NEISSERIA GONORRHOEAE CULTURE

## Orderable – GCCULT

Turnaround Time: 2 days

### Alternate Name(s):

GC Culture  
Gonorrhoeae Culture

### Specimen Type and Collection Information:



**Laboratory:**  
Microbiology



**Requisition:**  
Electronic Order



**Method of Analysis:**  
Microscopy, Culture



**Test Schedule:**  
Daily

Specimen Type	Collection Information	Collection Device
Cervical Swab Eye Swab Cyst Material Rectal Swab Throat Swab Urethral Meatus Swab Vaginal Swab (only for sexual assault)	Remove the swab and avoid touching the swab applicator below the pink scored breakpoint. Collect the sample by rolling the swab over the site. Insert the swab all the way into the transport tube and, holding the end of the swab applicator, bend it toward you until it breaks off. Cap the tube and label appropriately.	

### Storage and Shipment:

The specimen must be stored at room temperature.  
The specimen must be received within 24 hours of collection.