



Pathology and Laboratory Medicine

ALPHA-2-MACROGLOBULIN SERUM

Orderable – A2M

Turnaround Time: 20 days

Alternate Name(s):

A2M

Specimen:



Laboratory: Core Lab



Requisition:

GENERAL LABORATORY REQUISITION



Method of Analysis: Turbidimetry Adult Pediatric

5 mL Gold top
Vacutainer tube

Pediatric

2-10 years: 3 mL Gold top
Vacutainer tube

Collection Information:

Minimum volume of serum required is 1 mL.

Reference Ranges:

0.74 - 2.98 g/L



Test Schedule:

Referred out Monday-Thursday

Storage and Shipment:

Please ensure to aliquot and freeze the sample within 24 hours of collection and send frozen.



Referred Out Location: In-Common Laboratories