

LIPOPROTEIN (a)

Orderable - LIPOA

Turn Around Time: 20 days

Alternate Name(s):

Lpa
Lp(a)
Lipoprotein a
Lipoprotein A

Specimen:

4 mL Lavender (EDTA) top Vacutainer tube or 6 mL Red top Vacutainer tube

Collection Information:

Minimum volume of serum/plasma required is 1 mL.

Samples must be separated within 4 hours of collection.

Reference Ranges:

Greater than 30 mg/dL associated with 1.7-fold increased risk of cardiovascular disease

Less than 8.94 mg/dL lower limit for reporting
--

Storage and Shipment:

Store and send frozen.



Laboratory:
Core Lab



Requisition:
GENERAL LABORATORY
REQUISITION



Method of Analysis:
Siemens BN ProSpec N
Latex enhanced
nephelometric assay



Test Schedule:
Referred out Monday-
Thursday



Referred Out Location:
[In-Common Laboratories](#)