

SYSTEMIC SCLEROSIS PROFILE

Orderable - .ANANP

Turnaround Time: 5 days

Alternate Name(s):

Anti Scl-70
Anti CENP A
Anti CENP B
Anti RP11
Anti RP155
Anti Fibrillarin
Anti NOR90
Anti Th/To
Anti PM-Scl100
Anti PM-Scl75
Anti Ku
Anti PDGFR
Anti Ro-52



Laboratory:
Clinical Immunology Lab



Requisition:
GENERAL LABORATORY
REQUISITION



Method of Analysis:
Immunoblot



Test Schedule:
Batch analysis

Specimen:

Adult	Pediatric
5 mL Gold top Vacutainer tube or 6 mL Red top Vacutainer tube	0-2 years: Red 0.5 Microtainer 2-10 years: 2 mL Red top
Light Green (Li-Heparin) or Lavender (EDTA) top tubes are also acceptable	

Collection Information:


Minimum volume required for testing is 0.5 mL

Reference Ranges:

Negative

Interpretive Comments:

Immunoblot results will be reported as either negative or positive for specific

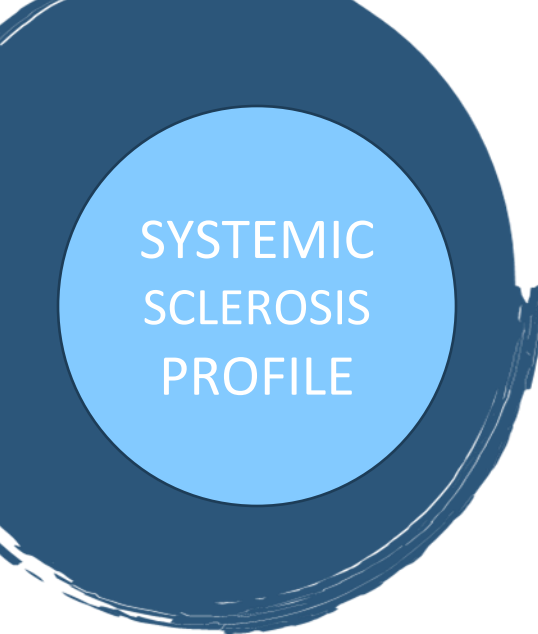

SYSTEMIC SCLEROSIS PROFILE

antibodies as the following:
(+) Borderline
1+ Weak Positive
2+ Positive
3+ Strong Positive

Comments:

This test is available exclusively to Rheumatologists and Respiriologists at LHSC/SJHC and accepted from referred in locations.

Specimen Stability:

Sample can be stored for up to 14 days at 4°C. For longer storage, store at -20°C.