

CYCLOSPORINE

Orderable – CYA/CYAC (Continuous dose)

Turnaround Time: 1 day

STAT: Same day if received in lab by 1000 h

Alternate Name(s):

CYA
Neoral®

Specimen:

Adult	Pediatric
4 mL K ₂ or K ₃ EDTA Lavender top Vacutainer tube	0-2 years: 0.5 mL Lavender Microtainer 2-10 years: 3 mL Lavender top

Collection Information:

Minimum volume required for testing is 1.5 mL.
Send to Laboratory immediately after collection
Clotted blood is not suitable for analysis.

Thoroughly mix the blood sample after procurement.

Reference Range:

No established reference range. Concentrations are measured in ng/mL.

Comments:

Call the Toxicology/Special Chemistry Laboratory for more information: 519-685-8500 x 64664 then choose option #3.

Critical Information Required:

Specify type of organ transplanted.
Date and time of sample collection.



Laboratory:
Toxicology/Special
Chemistry Lab




Requisition:
GENERAL LABORATORY
REQUISITION



Method of Analysis:
LC-MS/MS



Test Schedule:
Samples are tested
Monday-Saturday.
Specimens received in the
Core Laboratory after
1000 h will be processed
the next working day.

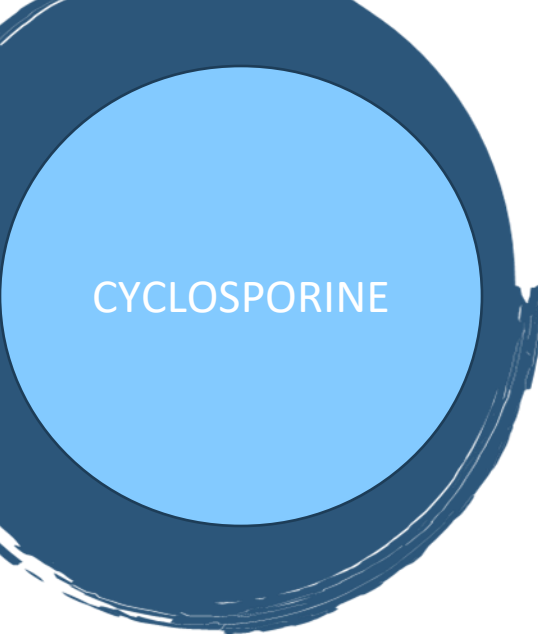

CYCLOSPORINE



London Health
Sciences Centre



ST. JOSEPH'S
HEALTH CARE
LONDON

Pathology and Laboratory Medicine

Storage and Shipment:

Within LHSC (between campuses):
Send samples on ice packs.

Referred-in:
Send on ice packs.