

HEPATITIS B ANTIVIRAL RESISTANCE

Orderable - HEPBAR

Turnaround Time: 1 week STAT: 3 days

Specimen:

Blood - 5 mL Gold Tube top Vacutainer (minimum 1 mL of serum)

Collection Information:

Collect blood aseptically in a Vacutainer tube.
Transport to the Lab STAT
Serum must be taken off the cells within 4 hours of collection.

Interpretive Comments:

See patient report for mutation information.

Special Processing:

Serum must be separated from the clot within 4 hours of collection and shipped frozen to the testing laboratory.

Critical Information Required:

The requisition must be filled out completely or testing will not be performed

Storage and Shipment:

The specimen must be refrigerated.
The specimen must be received within 24 hours of collection.



Laboratory:
Microbiology



Requisition:
[Hepatitis B Antiviral
Resistance Requisition](#)
This requisition MUST
accompany the specimen



Test Schedule:
Referred out weekdays



Referred to:
National Microbiology
Lab
1015 Arlington St.
Winnipeg, MB
R3E 3R2
Phone: 204-789-6062