

# How To Change the Date or Time - PrismaFLex



A password is not needed. You must change date and time before selecting new patient.

1. Turn on machine. Choose “Continue”.
2. Choose “Custom Mode”.
3. Choose “Continue”.
4. Choose “Time and Date”
5. Change the date and time. Select Hours or Minutes from bottom and then adjust.
6. When done, select “Exit Custom” (there is no password or requirement to confirm).
7. Confirm that the date/time is set correctly.

FLEX 5

CLEAN SCREEN WITH 70% ISOPROPYL ALCOHOL & SOFT CLOTH

prismaflex

26/March/25 16:45

Setup NoCh

## Prismaflex® System

Software Version: 7.21

**THERAPY INFO** - View information about:  
· Therapy possibilities.  
· Prismaflex Sets available for each therapy.  
· Access Choose Patient screen when ready.

**CONTINUE** - Proceed to Choose Patient screen:  
· Begin a patient setup.  
· Access history data of last treatment.

Copyright 2017 Gambro Lundia AB

THERAPY  
INFO

CONTINUE

# Step One

Turn on machine.

Choose **“Continue”**.

FLEX 5

CLEAN SCREEN WITH 70% ISOPROPYL ALCOHOL & SOFT CLOTH

prismatlex

## Step Two

Choose **“Custom Mode”**.

DO NOT choose patient or you will need to reboot to change clock/time.

Note, when setting up Flex for the same patient, choose “same patient”. This will pull up previous settings. It also allows you to view events that happened during the previous treatment(s) for this patient. The PRISMAX does not provide this option.

### Choose Patient

26/March/25 16:46

Setup NoCh

NEW  
PATIENT



- Set up control unit for a new patient.
- Delete treatment history data.
- Enter patient ID and weight.

SAME  
PATIENT



- Continue last treatment.
- Retain treatment history data.

CUSTOM  
MODE

- Modify default values.

THERAPY  
INFO

- View therapy possibilities.
- View Prismaflex Sets available for each therapy.

LAST  
HISTORY

- View history data of last treatment.

DOWNLD  
DATA

- Download history data of last treatment.

CUSTOM  
MODE

THERAPY  
INFO

NEW  
PATIENT

SAME  
PATIENT

LAST  
HISTORY

DOWNLD  
DATA

HELP



CLEAN SCREEN WITH 70% ISOPROPYL ALCOHOL & SOFT CLOTH

prismatix

## Welcome to Custom Mode

26/March/25 16:46

Custom NoCh

Caution: Custom Mode alters DEFAULT VALUES. Only authorized personnel should use Custom Mode.

Therapies:	Status:
I. CRRT(Continuous Renal Replacement therapies) including SCUF, CVVH, CVVHD, CVVHDF	Enable
II. TPE (Therapeutic Plasma Exchange)	Disable
III. HP (Hemopurification)	Disable
IV. CRRT septeX (High Cut Off) including CVVHD, CVVHD+post	Disable
V. CRRT MARS (Molecular Adsorbents Recirculation System) including CVVHD, CVVHDF	Disable

Note: Therapies can be enabled/disabled in Service mode only, by trained and qualified service technicians.

CONTINUE

EXIT  
CUSTOM

## Step Three

Choose **“Continue”**.

CLEAN SCREEN WITH 70% ISOPROPYL ALCOHOL & SOFT CLOTH

prismatix

## Step Four

Choose **“Time & Date”**.

### Modify Defaults

26/March/25 16:46

Custom NoCh

Softkey: Changes default values for:

FILTER  
SETTINGS

- Settings for each therapy/set combination, e.g. Alarm Limits and Flow Rates.

SYRINGE

- Syringe brand allowed for use.

BAG  
VOLUME

- Allowed Volume for bags/containers in each therapy/ fixed or variable Empty Bag method.

OTHER  
SETTINGS

- Chart/Doses Begin Time and Chart Interval for History screen  
- Status Graph Period.  
- Volume for Auto Blood Return

TIME  
& DATE

- Current Time and Date.  
- Date Display (day/mo/yr or mo/day/yr).

CANCEL

FILTER  
SETTINGS

SYRINGE

BAG  
VOLUME


OTHER  
SETTINGS

TIME  
& DATE

EXIT  
CUSTOM



CLEAN SCREEN WITH 70% ISOPROPYL ALCOHOL & SOFT CLOTH  
prismatlex

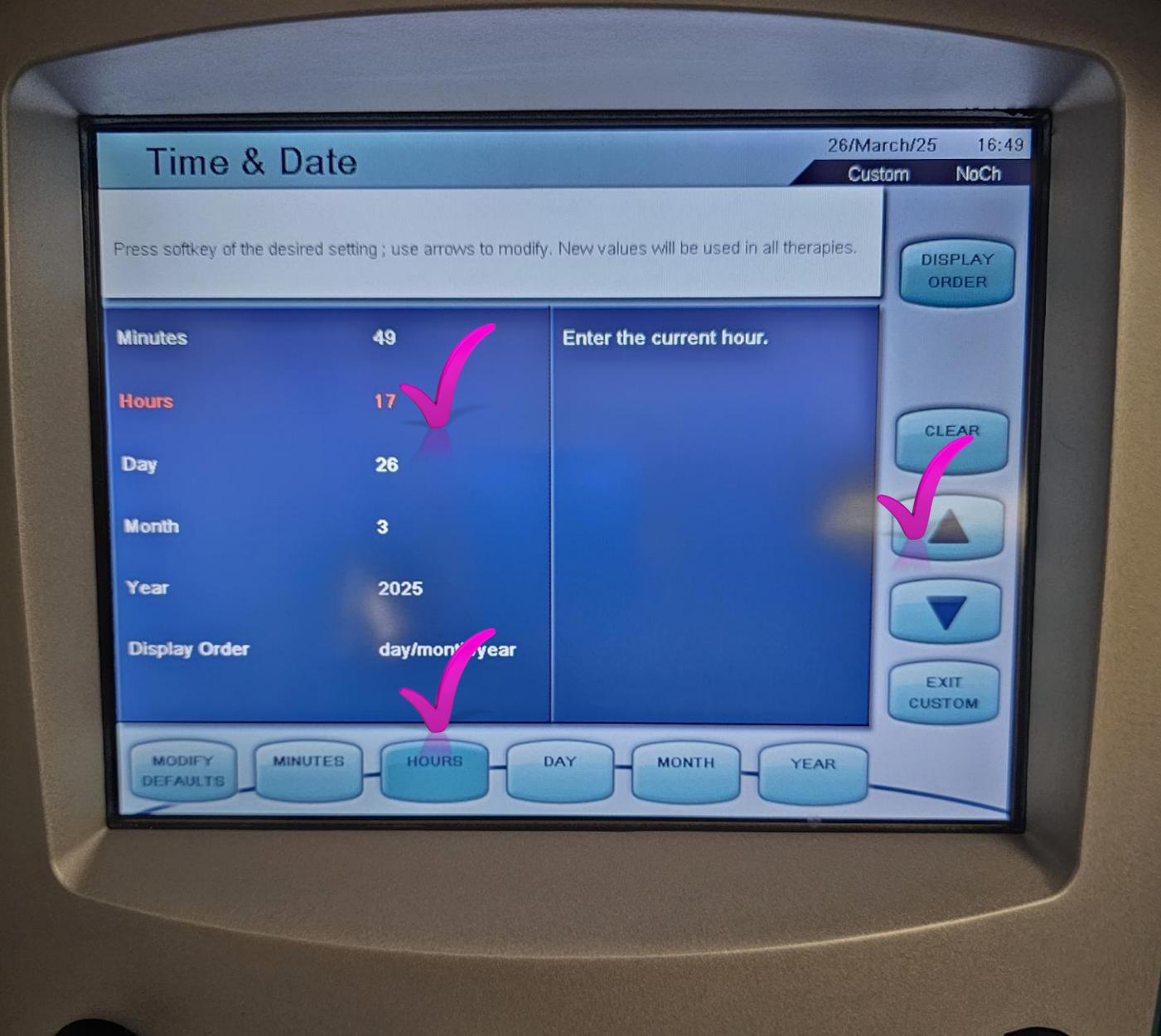


## Step Five

Choose desired parameter from bottom menu.

“Hour” has been selected from the bottom menu in this example. Note that “Hours” is now highlighted in colour, indicating the parameter that will be changed by adjusting the arrows to the right.

CLEAN SCREEN WITH 70% ISOPROPYL ALCOHOL & SOFT CLOTH  
prismatlex

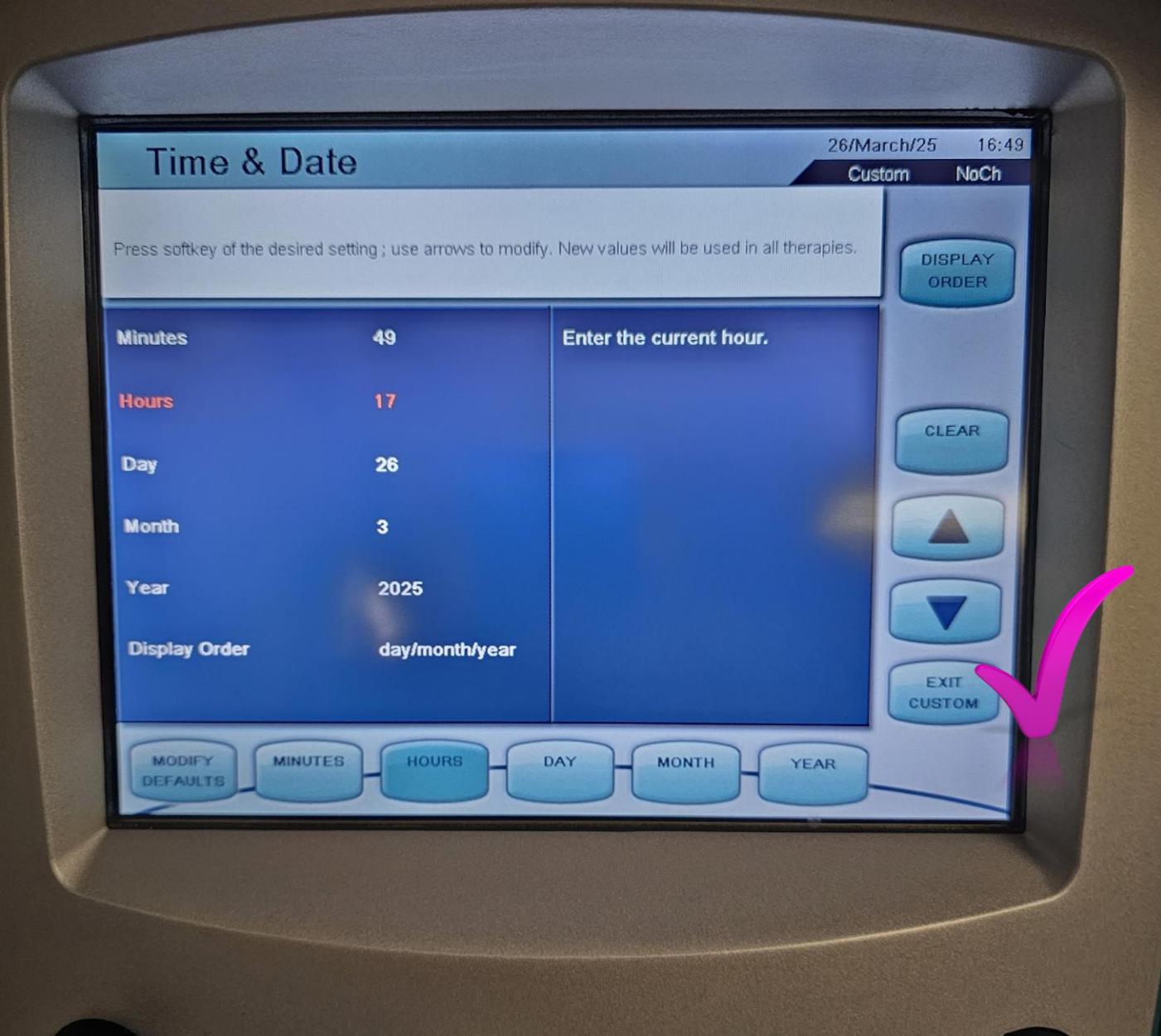

## Step Six

When done, select “Exit Custom”. This will save the last adjustment.

Unlike the PRISMAX, you do not have to enter a code or confirm the setting change (twice with PRISMAX).

FLEX 5

CLEAN SCREEN WITH 70% ISOPROPYL ALCOHOL & SOFT CLOTH

prismatlex



# Choose Patient

26/March/25 17:49

Setup NoCh

- NEW PATIENT**
  - Set up control unit for a new patient.
  - Delete treatment history data.
  - Enter patient ID and weight.
- SAME PATIENT**
  - Continue last treatment.
  - Retain treatment history data.
- CUSTOM MODE**
  - Modify default values.
- THERAPY INFO**
  - View therapy possibilities.
  - View Prismaflex Sets available for each therapy.
- LAST HISTORY**
  - View history data of last treatment.
- DOWNLD DATA**
  - Download history data of last treatment.

CUSTOM MODE   THERAPY INFO   NEW PATIENT   SAME PATIENT   LAST HISTORY   DOWNLD DATA   HELP

# Step Seven

Confirm that the time was correctly changed.