

How To Change the Date or Time - PRISMAX




1. Turn on machine.

DO NOT select New Patient. Date/time cannot be changed once new patient is selected.

Select “System” from top menu.

2. On the right side of screen, select drop down arrow beside “Operator”
3. Choose “Site Expert”
4. Enter the Code that was sent to your EMAIL, then select blue checkmark
5. Go to “Display” to the left of the screen. Select down down arrow and choose “Date and Time”.
6. Adjust the date or time, then select “APPLY” below the time selections.
7. Select “ACCEPT” at bottom right of screen.
8. Confirm that the date/time was correctly changed.

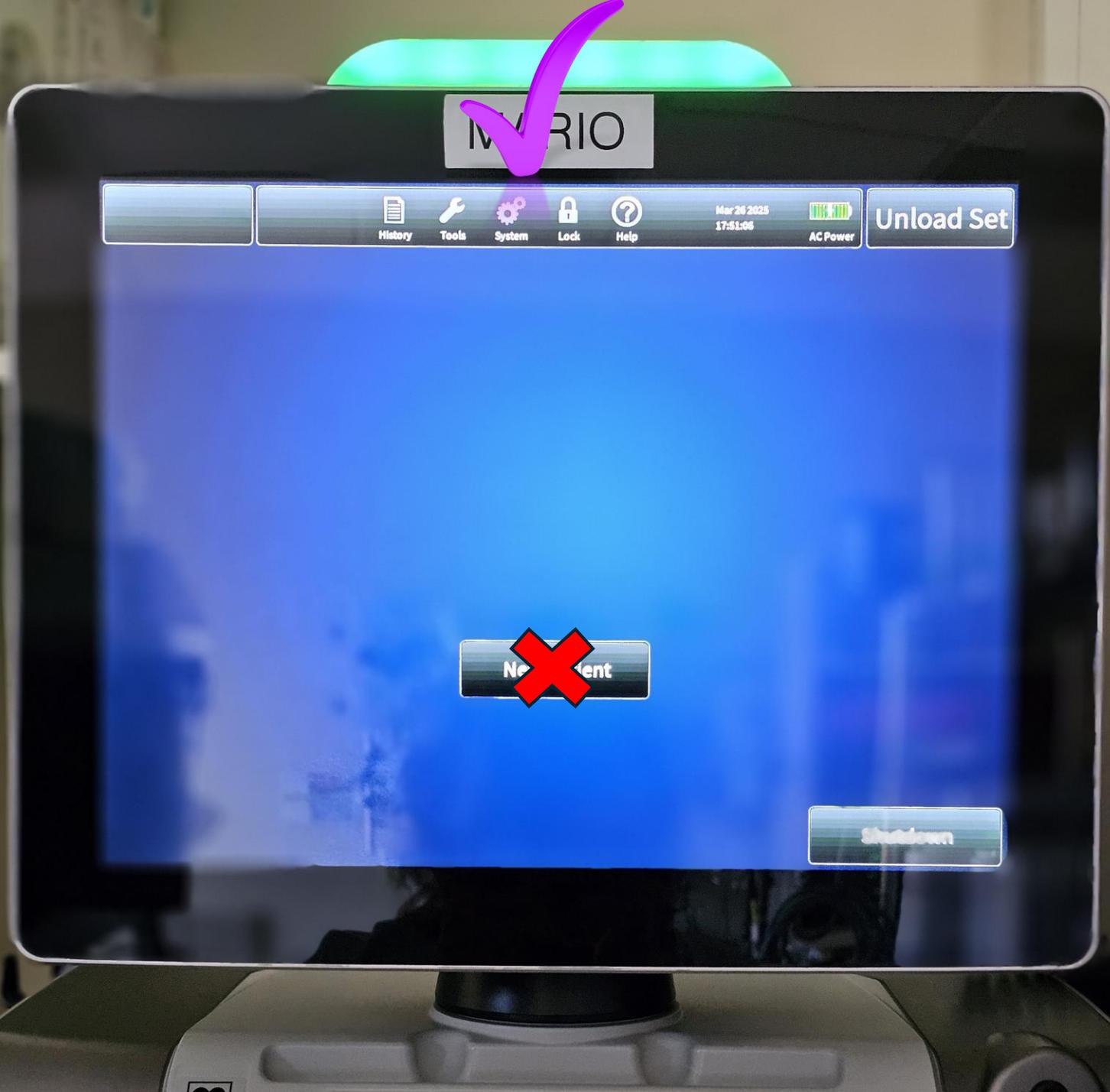

Step One

Turn on machine.

When you reach this screen, DO NOT select New Patient.

Choose **“System”**.

MARIO

History Tools System Lock Help Mar 26 2025 17:52:01 AC Power Unload Set

System Configuration

Operator

Display

Set Display Brightness

Sound

System Info

USB Import/Export


Status Light
Bright
Dim

Cancel

Accept

Step Two

Select drop-down arrow beside **“Operator”** (right upper screen).

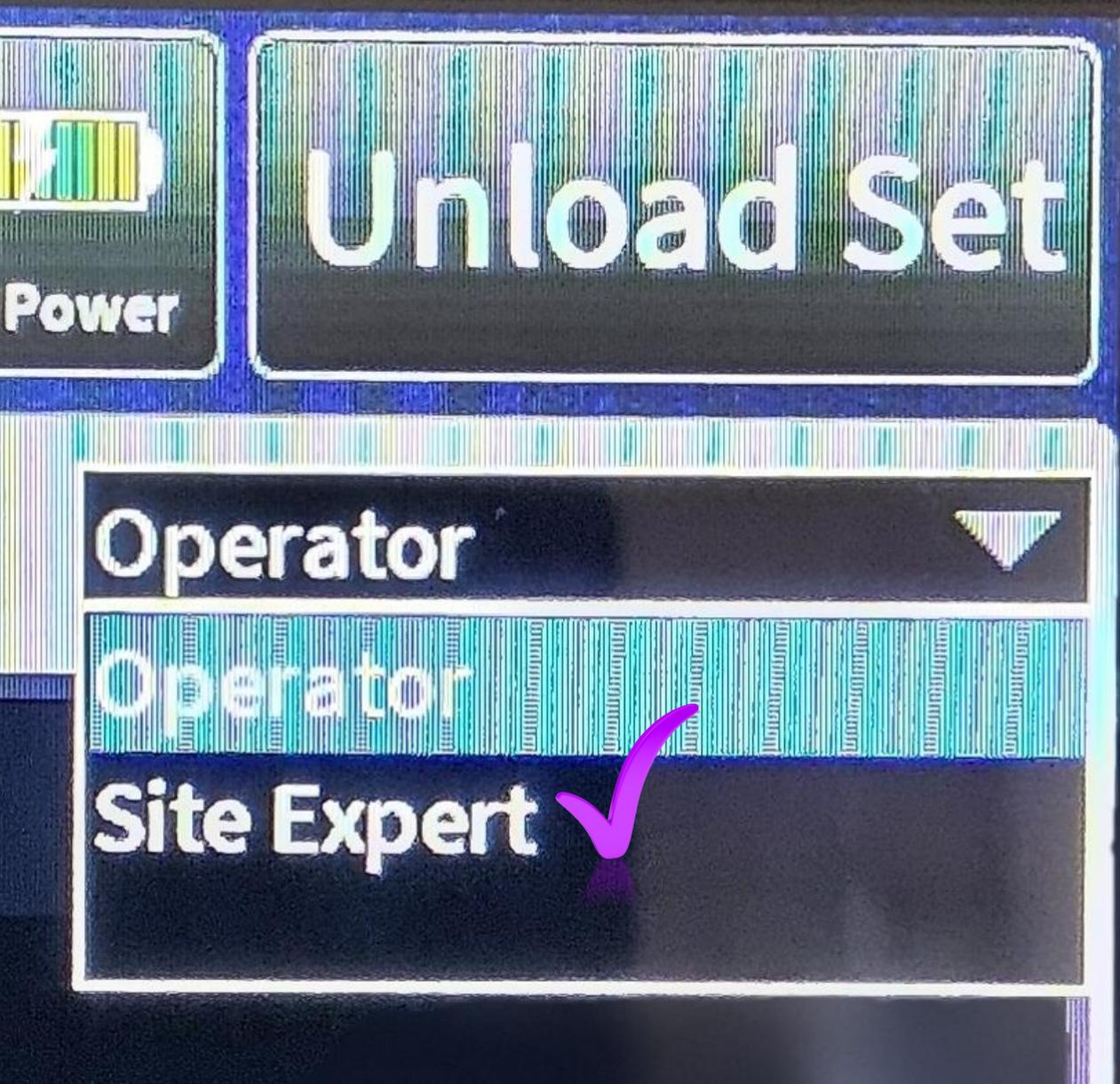

Step Three

Select **“Site Expert”**.

Mar 26 2025
17:52:38



Unload Set

Site Expert




0 999999



Step Four

Enter the 6 number code that was sent to you by email.


When done, select the blue **checkmark**.

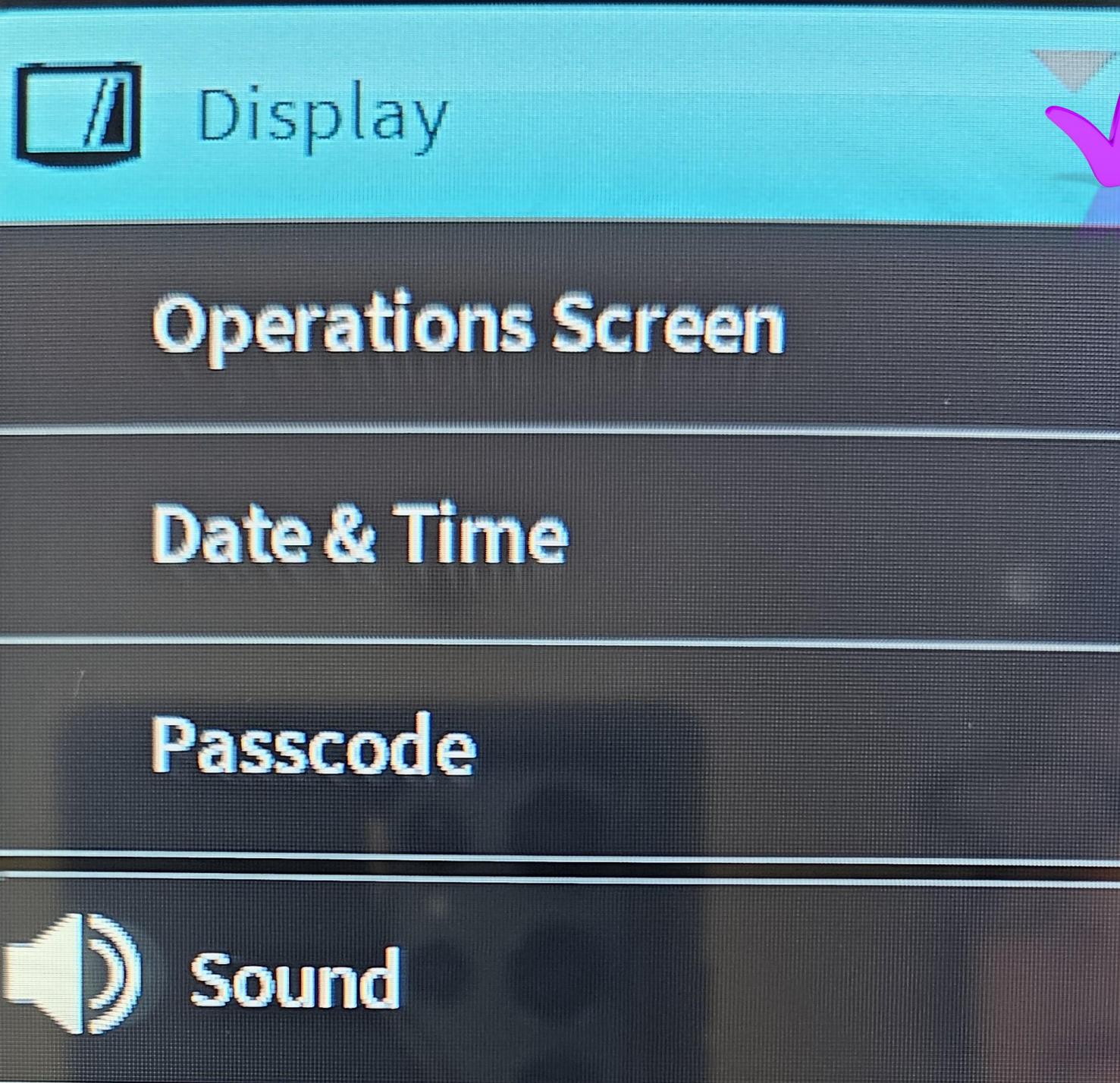

Step Five

To the right of the screen, choose the drop-down arrow beside **“Display”**.

Select **“Date & Time”**.

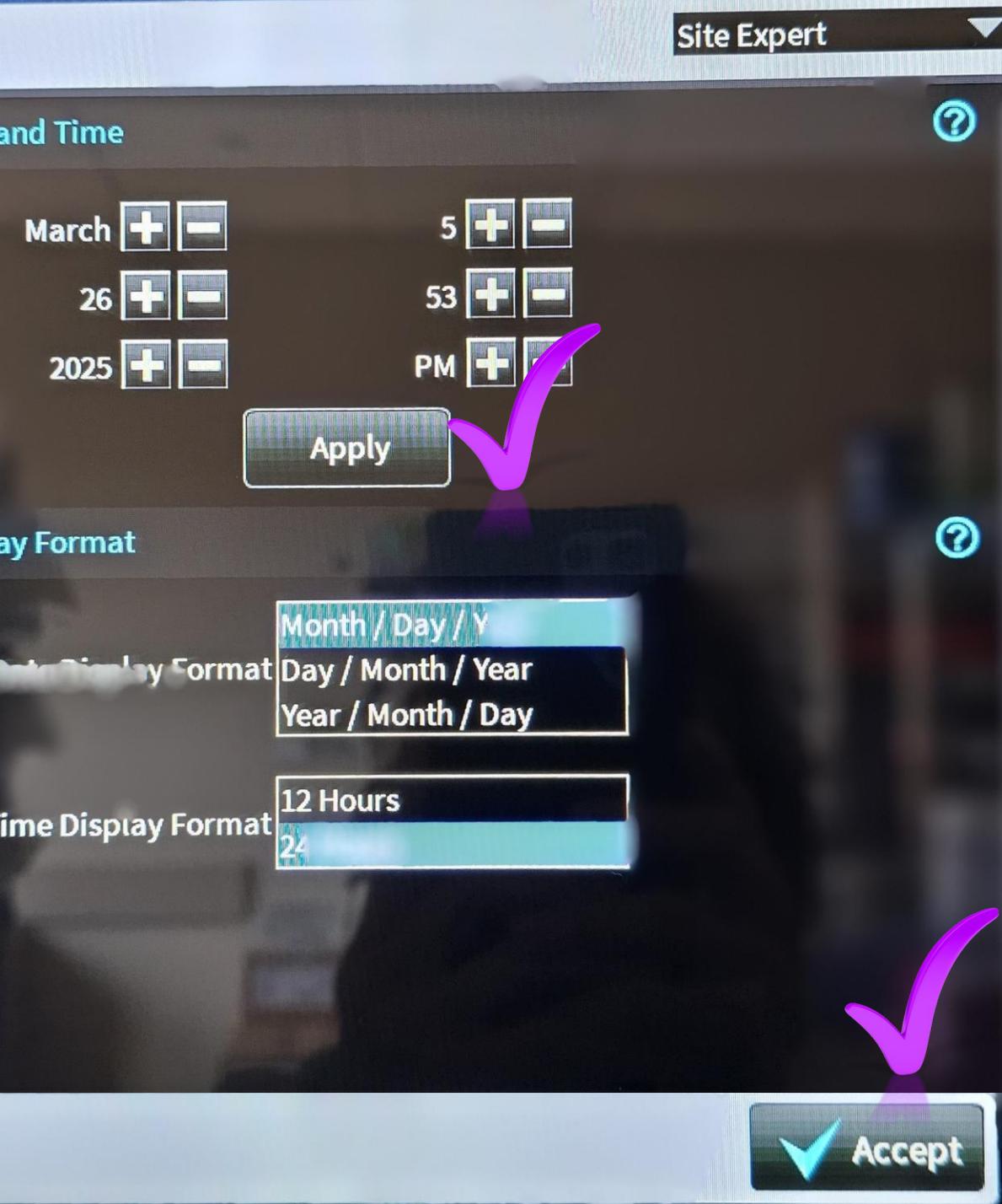

Step Six and Seven

Adjust date and time. Leave the format as 24 hours.

Note that the “5” refers to 5 PM, which will be displayed at 1700 on the machine.

Select “**APPLY**”.

Confirm the new setting by selecting “**ACCEPT**” at the bottom.

Step 8: Confirm Change

Confirm that the date and time was correctly changed. If the time did not update, go back and make sure you double confirm (select apply followed by accept).

