


New Stopcock for CRRT Rescue Line (replaces “Y” Connector)

Smith Medical (Medex) MX5341L HMMS

1. **This is a 4-way stopcock (like ICP stopcock)** – ensure positioned to only allow flow in 2 directions .
2. This is a high flow stopcock used on ECMO circuits.
3. The stopcock swivels when connected correctly – it doesn’t mean it is loose.
4. See next page for correct connection.
5. Keep the slide and roller clamp on saline open at all times and control flow rate through stopcock only for quick emergency retransfusion with minimal alarms.



Operating Position

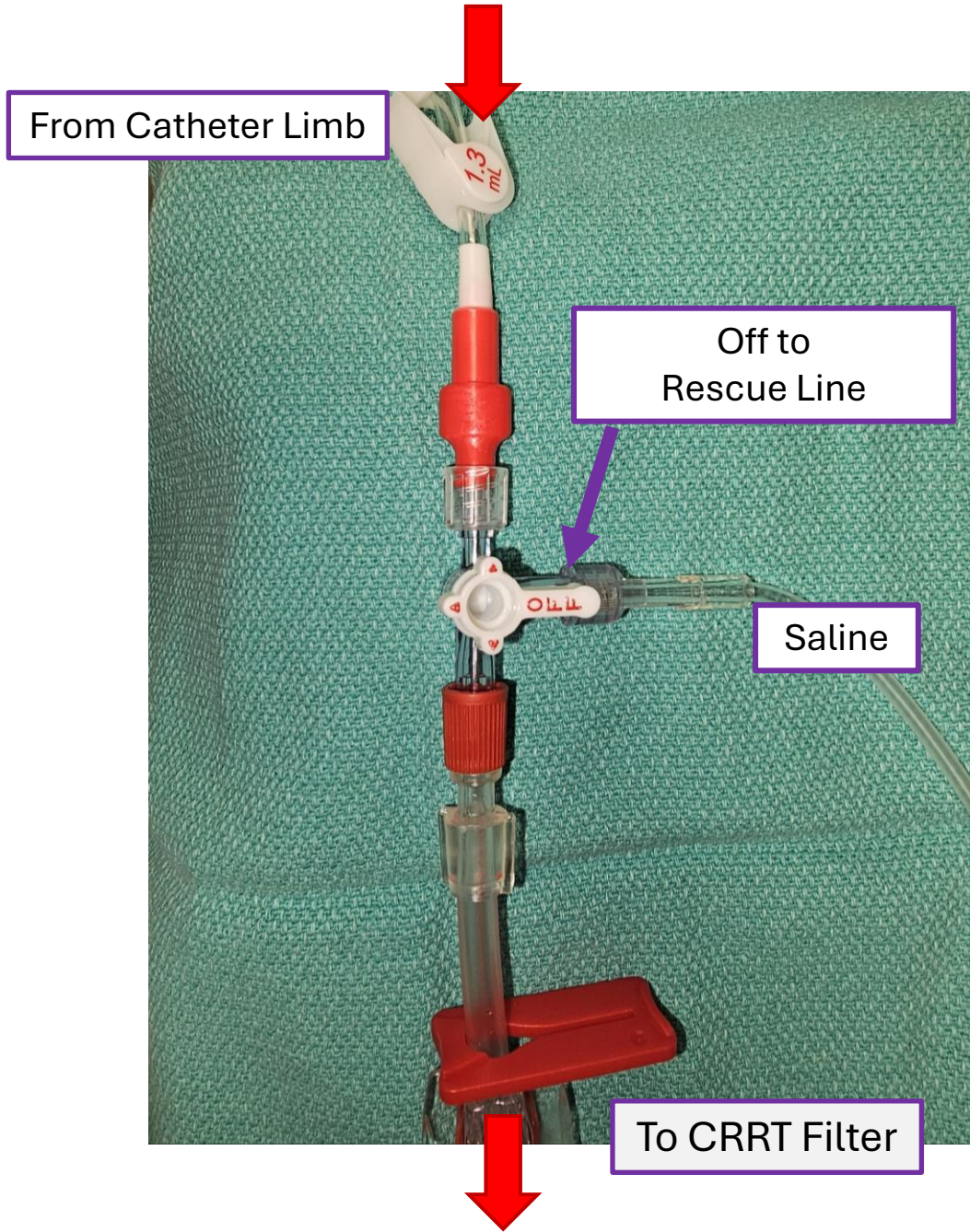

Retransfusion Position

