

Changing Profiles


Use the Adult (default) or Comfort Profiles for all patients.

The only difference in the CSRU profile is that the pacemaker defaults to “ON”. Turn pacemaker on in the Adult Profile if indicated.

Changing Profiles

ANY TIME YOU CHANGE A PROFILE, you will revert to the original default settings for all waveforms and alarms. **Instead of changing the profile, modify your screen. The modified version of a screen will have an asterisk (*).**

You can change back and forth between the default and modified screen.

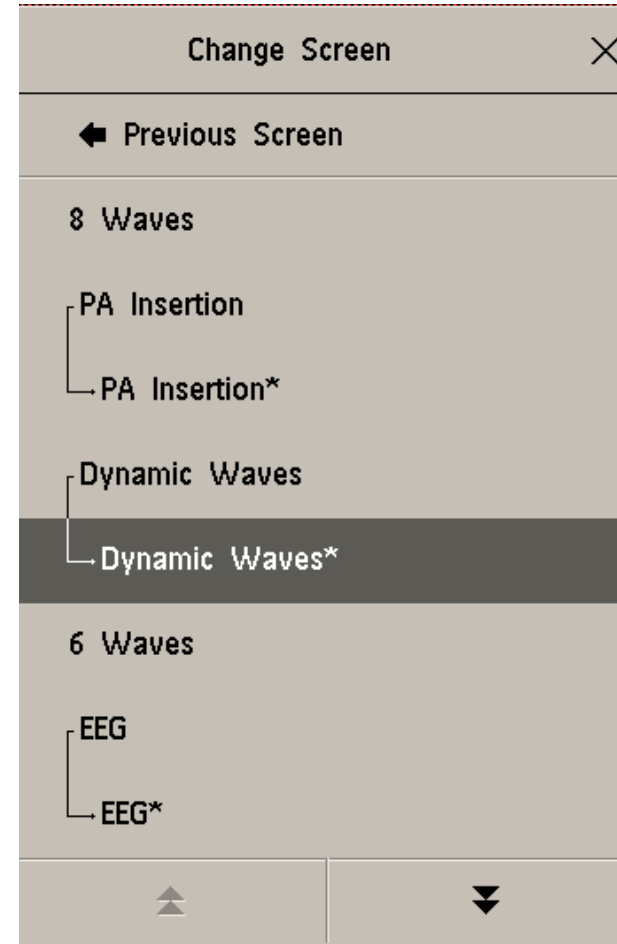

Comfort Mode

YOU MAY NOW USE THE COMFORT PROFILE.
Touch the Profile .“Adult” is shown here. Choose
Comfort Profile. **This will turn all alarms OFF.**



Using Comfort Screen




Future changes will include disabling of the alarm and the replacement of the “Clock” with a butterfly.



Turning All Alarms Off


During WLS or a resuscitation where the team is present and the patient is triggering continuous alarms, you need to turn all alarms OFF.

1. Go to Alarm limits (from Smart Keys)

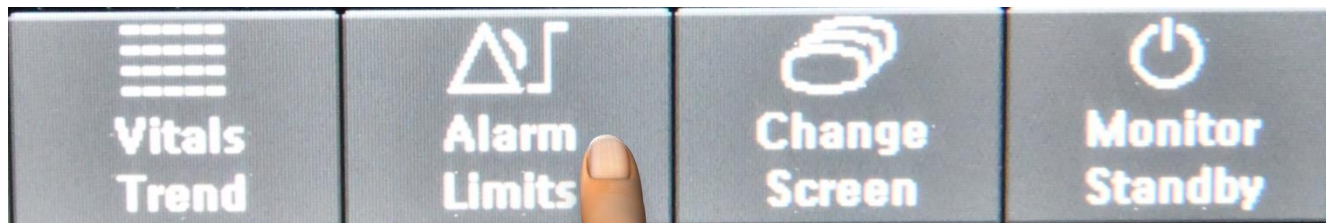

Turning All Alarms OFF

2. Choose “All Alarms Off”.

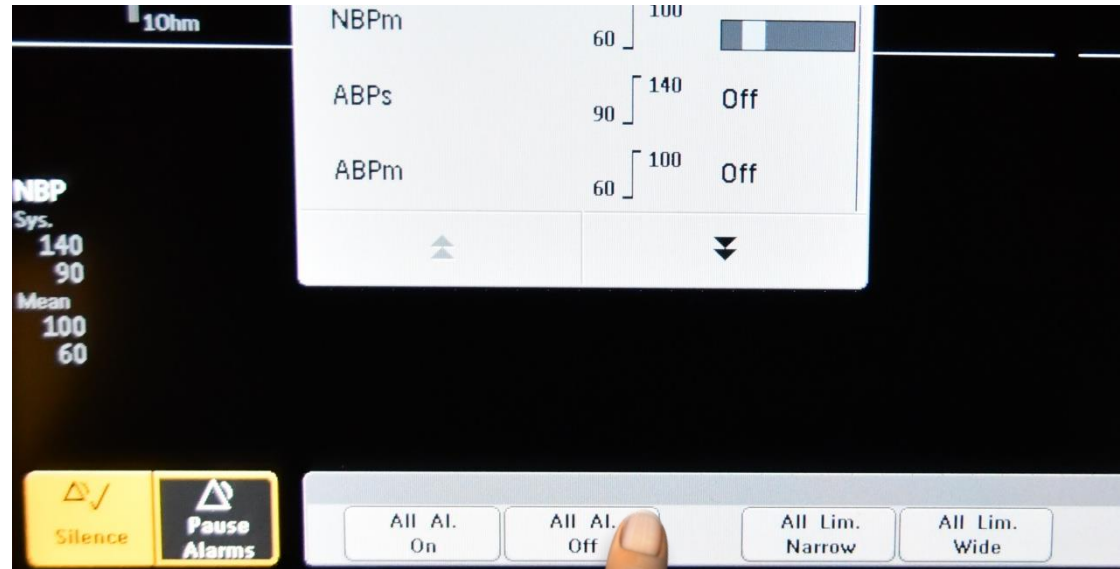

Turning All Alarms OFF

3. Confirm from the menu at the bottom. When you confirm, you are assuming responsibility for the disabled alarms (document the reason the alarms are OFF).

