

CREATININE  
24-HOUR  
URINE

## Orderable - CREU24

Turn Around Time: 24 hours

STAT: 1 hour

### Specimen:

24 Hour Urine

### Collection Information:

Collection of 24-Hour Urine Sample

Collect urine over a 24-hour period as per container label instructions.

No preservative necessary, but specimen should be kept refrigerated during collection.

Acidified or alkaline urine is acceptable for analysis.

Please complete the laboratory label attached to the collection bottle to include patient's name, collection start time and date and collection end time and date.

### Reference Ranges:

Male	Female
8.6-19.4 mmol/d	6.3-13.4 mmol/d



**Laboratory:**  
Core Lab



**Requisition:**  
GENERAL LABORATORY  
REQUISITION



**Method of Analysis:**  
Enzymatic method



**Test Schedule:**  
Monday - Friday  
0800 - 1600 h