

ESTRONE,  
SERUM/  
PLASMA

## Orderable - ESTRO

Turn Around Time: 30 days

### Alternate Name(s):

E1

### Specimen:

5 mL Gold top Vacutainer tube

### Collection Information:

Minimum volume of serum or plasma (Heparin) required is 1 mL.

### Reference Ranges:

Male	
105-275 pmol/L	

Female	
Follicular Phase:	144 - 487 pmol/L
Luteal Phase:	200 - 663 pmol/L
Peri-ovulatory:	214 - 947 pmol/L
Post-menopausal:	
With Estrogen HRT:	189-1803 pmol/L
Without Estrogen HRT:	114-370 pmol/L

### Special Processing:

Store and send frozen.



**Laboratory:**  
Core Lab



**Requisition:**  
GENERAL LABORATORY  
REQUISITION



**Method of Analysis:**  
Radioimmunoassay (RIA)



**Test Schedule:**  
Referred out Monday-  
Thursday



**Referred Out Location:**  
[In-Common Laboratories](#)